INSULIN PEN DEVICES GUIDE



TOP TIPS:

- Prescribe/use different colour of pen device for each type of insulin
- Prescribe one spare device for each type of insulin used (as replacement in emergency)
- Add "to be replaced by date" within dosage instructions to multiple-use pen devices
- All pen devices below are compatible with GlucoRx CarePoint or CarePoint Ultra pen needles (cost effective option) or any other brand of screw-on insulin pen needles

DEVICE NAME	MADE BY	APPEARANCE (other colours are available)	DOSAGE (min-max)	INSULIN used in pen	LIFESPAN (to be replaced after)	*Rx ISSUED
Humapen SAVVIO	Eli Lilly		1-60 units	Lilly 3ml cartridges from Humalog and Humulin ranges	6 years	А
Humapen LUXURA HD	Eli Lilly	DISCONTINUED by manufacturer - as alternative use pre-filled Junior KwikPen	0.5-30 units (half unit increments)	Lilly 3ml cartridges from Humalog and Humulin ranges	3 years	n/a
KwikPen pre-filled	Eli Lilly	F	1-60 units	Pre-filled pens Humalog, Humalog Mix25, Humalog Mix50, Abasaglar, Lyumjev, Humulin M3 and Humulin I	n/a (discard 28 days after first use)	R
Junior KwikPen pre-filled	Eli Lilly	Control of the Contro	0.5-30 units (half unit increments)	Pre-filled pens with Humalog or Lyumjev only	n/a (discard 28 days after first use)	R
NovoPen 6	Novo Nordisk	Noosher' 6 9	1-60 units (with memory function & integrated with selected apps)	Novo Nordisk 3ml Penfill cartridges Insulatard, Levemir, NovoMix 30, NovoRapid, Fiasp and Tresiba	4-5 years (depending on battery life and memory capacity)	А
NovoPen Echo Plus	Novo Nordisk	Nowher Editor Pleas	0.5-30 units (half unit increments, with memory function & integrated with selected apps)	Novo Nordisk 3ml Penfill cartridges Insulatard, Levemir, NovoMix 30, NovoRapid, Fiasp and Tresiba	4-5 years (depending on battery life and memory capacity)	А
FlexPen pre-filled	Novo Nordisk	Turber*	1-60 units (audible clicks)	Pre-filled pens NovoRapid, Levemir, NovoMix 30	n/a (discard 28 days after first use)	R
FlexTouch pre-filled	Novo Nordisk	RexTouch*	1-80 units (100units/1ml, 1 unit increment, audible clicks) 2-160 units (200units/1ml, 2 unit increment, audible clicks)	Pre-filled pens NovoRapid, Tresiba U100, Fiasp Pre-filled pens Tresiba U200	n/a (discard 28 days after first use)	R
InnoLet pre-filled	Novo Nordisk	G	1-50 units	Pre-filled pens Insulatard or Levemir only	n/a (discard 28 days after first use)	R
AllStar	Sanofi		1-80 units	Sanofi 3 ml cartridges	3 years	Α

JuniorSTAR	Sanofi	JuniorSTAR - 0	1-30 units (half unit increments)	Sanofi 3 ml cartridges	2 years	А
SoloStar pre-filled	Sanofi	Lantus SoloStar 199 - 19	1-80 units	Pre-filled pens Lantus, Apidra, Admelog, Trurapi, Toujeo, Insuman Basal, Insuman Comb 25	n/a (discard 28 days after first use)	R
DoubleStar Pre-filled	Sanofi	Topical Docuber 1497	1-160 units	Pre-filled pens Toujeo	n/a (discard 28 days after first use)	R
Autopen Classic	Owen Mumford	Autopen Classic, 1–21 units in increments of 1 unit Autopen Classic, 2–42 units in increments of 2 unit	1-21 units (1 unit increments) 2-42 units (2 unit increments)	Lilly 3ml cartridges from Humalog and Humulin ranges, Wockhardt 3ml cartridges	3 years	А
Autopen24	Owen Mumford	Autopen 24, 2-42 units in increments of 2 unit Autopen 24, 1-21 units in increments of 1 unit	1-21 units (1 unit increments) 2-42 units (2 unit increments)	Sanofi 3 ml cartridges	3 years	А

*Rx issued: A - acute R - repeat